


It is essential to take accurate measurements to ensure the patient gets the correct levels of compression. To select the right style and size of stocking see the following:

To Measure Below Knee Style

- Ⓐ Measure the ankle circumference at the greatest point.
- Ⓑ Measure the calf circumference at the greatest point.
- Ⓒ Measure the distance from behind the knee down to the heel.

To Measure Thigh Length Style

- Ⓐ Measure the ankle circumference at the greatest point.
- Ⓑ Measure the calf circumference at the greatest point.
- Ⓒ Measure the thigh circumference at the greatest point.
- Ⓓ Measure the distance from the buttock fold down to the heel.

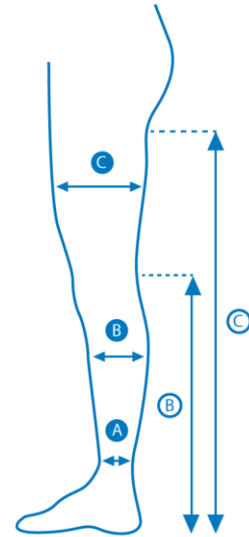

Saphena® Medical Anti-Embolism Stockings Thigh Length				
Size	Ankle Measurement	Calf Measurement	Thigh Measurement	Thigh Length
Small	19 - 23 cm	29 - 35 cm	Not Exceeding 65 cm	> 78 cm
Standard	23 - 26 cm	36 - 41 cm	Not Exceeding 70 cm	> 78 cm
Large	26 - 29 cm	42 - 47 cm	Not Exceeding 78 cm	> 78 cm
Extra Large	29 - 33 cm	48 - 54 cm	Not Exceeding 82 cm	> 78 cm

Saphena® Medical Anti-Embolism Stockings Below Knee			
Size	Ankle Measurement	Calf Measurement	Calf Length
Small Short	19 - 23 cm	29 - 35 cm	< 35 cm
Small	19 - 23 cm	29 - 35 cm	> 35 cm
Standard	23 - 26 cm	36 - 41 cm	> 35 cm
Large	26 - 29 cm	42 - 47 cm	> 35 cm
Extra Large	29 - 33 cm	48 - 54 cm	> 35 cm
Plus XX Large	33 - 35 cm	55 - 60 cm	> 35 cm